


# Quick Reference Guide

V3402

FOR VETERINARY USE ONLY

# SurgiVet™

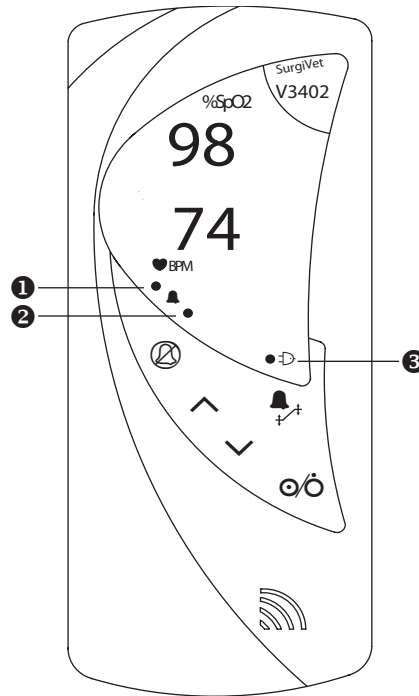


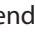
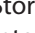



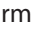
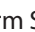






⚠ FOR COMPLETE INSTRUCTIONS, PLEASE REFER TO THE OPERATIONS MANUAL




Use the Optional Infrared Printer:  
MCP8850B, Catalog Number 8411

KEY	DESCRIPTION	INSTRUCTION
	Alarm Silence	<ul style="list-style-type: none"> <li>To silence an alarm for two minutes: Press  for a moment.</li> <li>To silence an alarm indefinitely: Press  for three seconds or more.</li> <li>To enable the audible alarm: Press  for a moment.</li> </ul> <p><i>Note: The indefinite alarm silence feature is not available for all countries.</i></p>
	Up/Down Arrows	To adjust a value or to select an item.
	On/Off	Turns the monitor On or Off.
	Menu/Enter	To display menus, enter menu selections or enter adjusted values.
	Print/ID/Clear	<p>Located on the side of the monitor.</p> <ul style="list-style-type: none"> <li><b>To print real time (every five seconds) to the optional IR printer:</b> Connect the SpO<sub>2</sub> sensor to the patient, then press  one time.</li> <li><b>To print relative time trend data to the optional IR printer:</b> Disconnect the SpO<sub>2</sub> sensor from the monitor, then press  one time.</li> <li><b>To stop printing:</b> Press  and hold for three seconds.</li> <li><b>To clear all trend data:</b> Press  for three seconds, [CLR] will flash.</li> <li><b>To reset the patient number and clear all trend data:</b> Press  for six seconds, [CLR] will flash and then [P1] will be displayed.</li> </ul> <p><i>Note: If trend data was stored for the previous patient, the patient number will increase by one, each time the monitor is turned off and then on again.</i></p>
	AC Power Jack	<p>Located on the side of the monitor.</p> <ul style="list-style-type: none"> <li>To connect an optional AC power supply.</li> </ul>
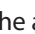
# smiths medical

MEASUREMENT	DESCRIPTION	INSTRUCTION
♥bpm	Pulse Rate	Displays a continuous, real-time update of the patient's pulse rate.
%SPO <sub>2</sub>	Percent of Oxygen Saturation	Displays a continuous, real-time update of the patient's %O <sub>2</sub> saturation.
INDICATOR	DESCRIPTION	INSTRUCTION
①	Alarm Indicator High Priority	Flashes RED when a patient's SpO <sub>2</sub> and/or Pulse Measurement matches or exceeds the high or low alarm limit for that measurement.
②	Alarm Indicator Low Priority	Shows steady YELLOW if: <ul style="list-style-type: none"> <li>• A condition exists preventing the monitor from taking a measurement.</li> <li>• The battery is low (with only thirty minutes remaining).</li> </ul>
③	Power Indicator 	<ul style="list-style-type: none"> <li>• If external power is on: Shows steady GREEN.</li> <li>• If the battery is charging: Flashes GREEN and blinks slowly.</li> </ul> <i>Note: This monitor uses six AA alkaline batteries, IEC Type LR6 (24 hour life supply).</i>
MONITOR SETUP	MENU	INSTRUCTION
<p>To avoid having to turn the monitor off and on, each time a monitor setting needs to be adjusted, please follow the instructions listed below, in the order that they are written.</p> <p><i>Note: Even after the monitor is turned Off, adjusted settings are stored until they are changed again.</i></p> <p><i>Note: Press  while turning on the monitor, and again to scroll through the menus if further changes are necessary.</i></p>		
At time of turning the monitor on.	Mode of Operation	<ol style="list-style-type: none"> <li>1. Press and hold  while turning on the monitor.</li> <li>2. [nor] will be displayed, do not adjust, [nor] is for normal mode.</li> </ol>
	Trend Data Storage Interval	<ol style="list-style-type: none"> <li>1. Press  again, to access the Trend menu.</li> <li>2. Press  to adjust the storage interval (4-30 seconds).</li> </ol>
	Alarm Volume	<ol style="list-style-type: none"> <li>1. Press  again, to access the Alarm Volume menu.</li> <li>2. Press  to adjust the audible alarm volume (3-15).</li> </ol> <p><i>Note: The alarm volume always supersedes the pulse volume.</i></p> <p><b>NOTE! The alarm volume cannot be turned off.</b></p>
	Low Priority Alarm Delay	<ol style="list-style-type: none"> <li>1. Press  again, to access the Low Priority Alarm Delay menu.</li> <li>2. [dl oFF] will be displayed.</li> <li>3. Press  to adjust the delay interval (15, 30, or 45 seconds).</li> </ol> <p><i>Note: The Low Priority Alarm Delay can be used during an SpO<sub>2</sub> search or an artifact event.</i></p>
	Alarm Silence Menu	<ol style="list-style-type: none"> <li>1. Press  again, to access the Alarm Silence menu.</li> <li>2. Press  to select [PrAUd] which allows permanent silence or [2AUd] which allows a two minute silence ( still needs to be pressed to activate).</li> </ol>
Alarms	Set the alarm limits for SpO <sub>2</sub> and Pulse Rate by pressing  to scroll through the alarm limit menu (SpO <sub>2</sub> HI/LO, then Pulse Rate HI/LO). <i>Note: High and Low alarm settings can be turned off for both the %SpO and Pulse Rate measurements.</i>	
Averaging	The default for averaging is 8/8 (8 SpO <sub>2</sub> , 8 sec. pulse rate), to select an averaging other than default: <ul style="list-style-type: none"> <li>• <b>16/16:</b> Press and hold  while turning on the monitor (16 SpO<sub>2</sub>, 16 sec. pulse rate).</li> <li>• <b>4/8:</b> Press and hold  while turning on the monitor (4 SpO<sub>2</sub>, 8 sec. pulse rate).</li> </ul>	
Pulse Beep Volume	Press  to adjust the Pulse beep volume. <i>Note: The Pulse beep volume can be turned Off.</i>	

© 2008 Smiths Medical Family of companies. All rights reserved.

SurgiVet and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol  indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.